



Quick Grapher


Easily explore and share your Kenek Edge data

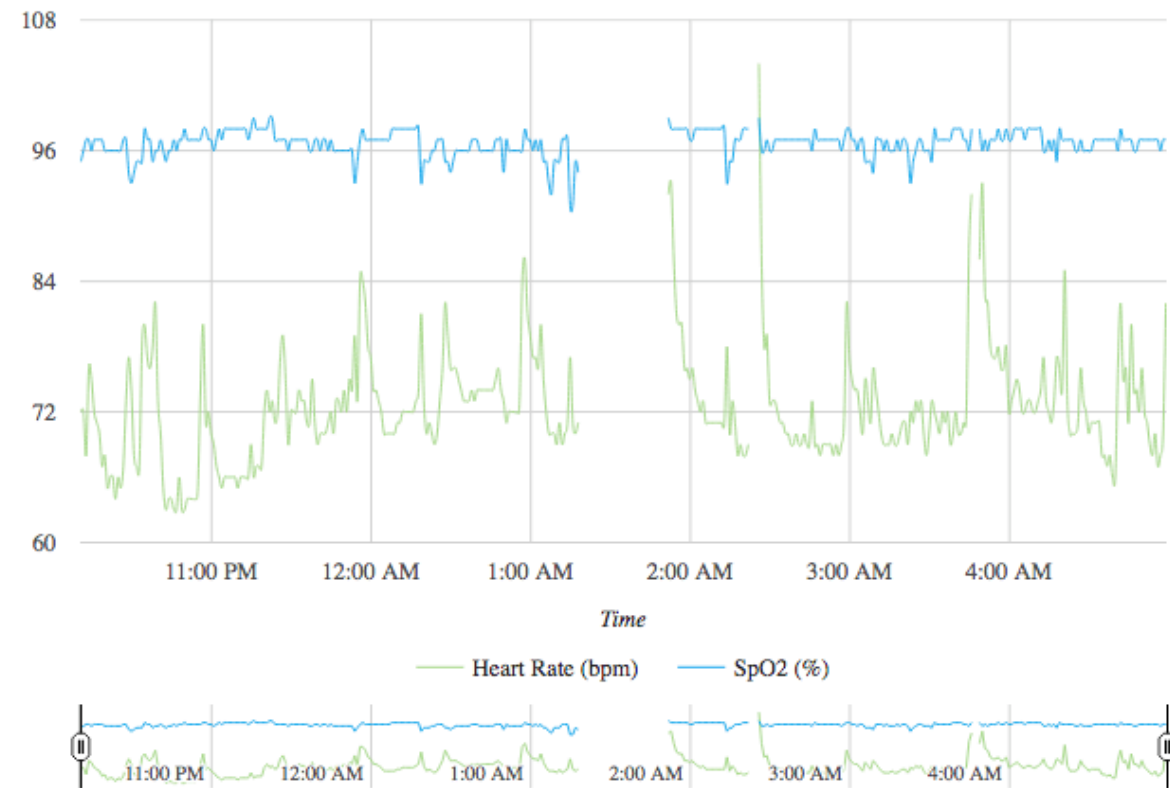


SpO2 Cumulative Distribution

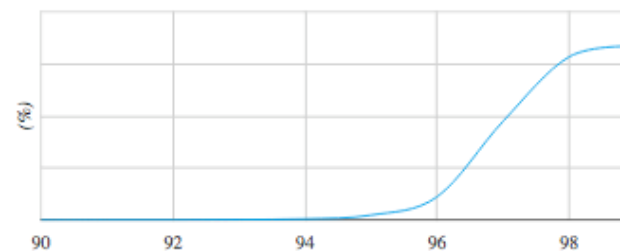

SpO2 Probability Distribution

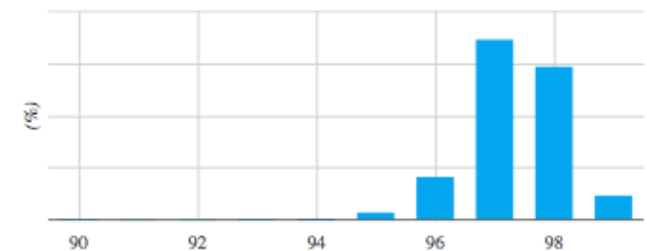


Heart Rate Cumulative Distribution

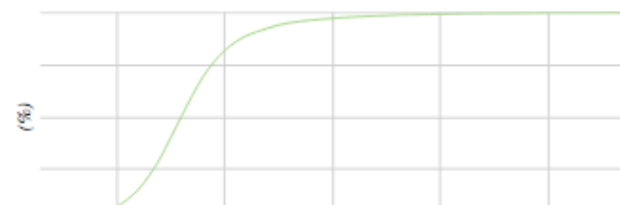

Heart Rate Probability Distribution



Oximeter

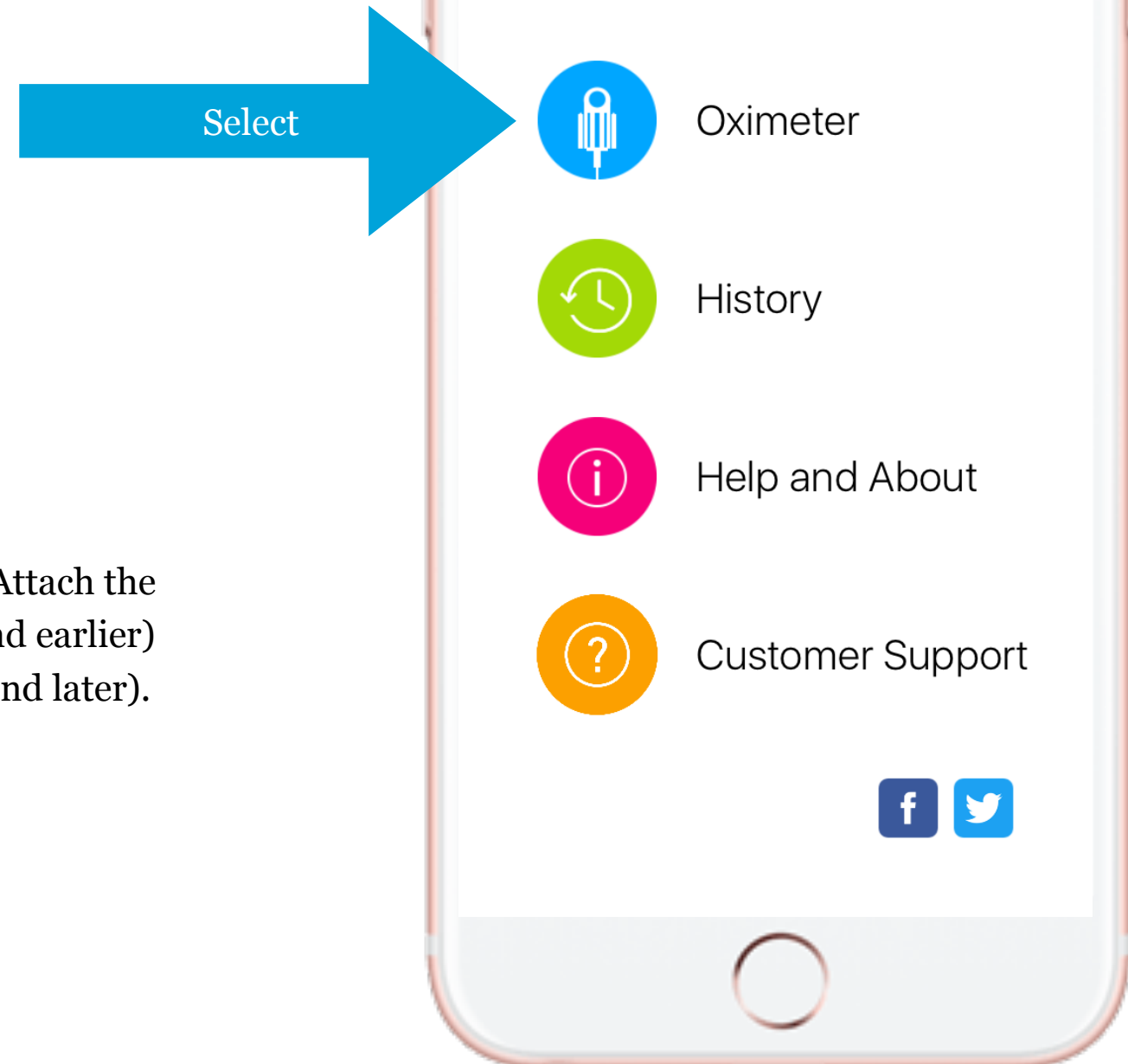


Open the Kenek Edge app on your iPhone or iPad. Attach the finger sensor to the audio port directly (iPhone 6 and earlier) or use an Apple provided audio adaptor (iPhone 7 and later). Select the Oximeter option from the home screen.



Audio Adaptor

An audio adaptor provided by Apple is required for newer iPhone models. Apple includes the adaptor in the packaging of the device. Adaptors are also available from the Apple store and online.

Plug the finger probe sensor into the adaptor then the adaptor into the iPhone's lightning port.

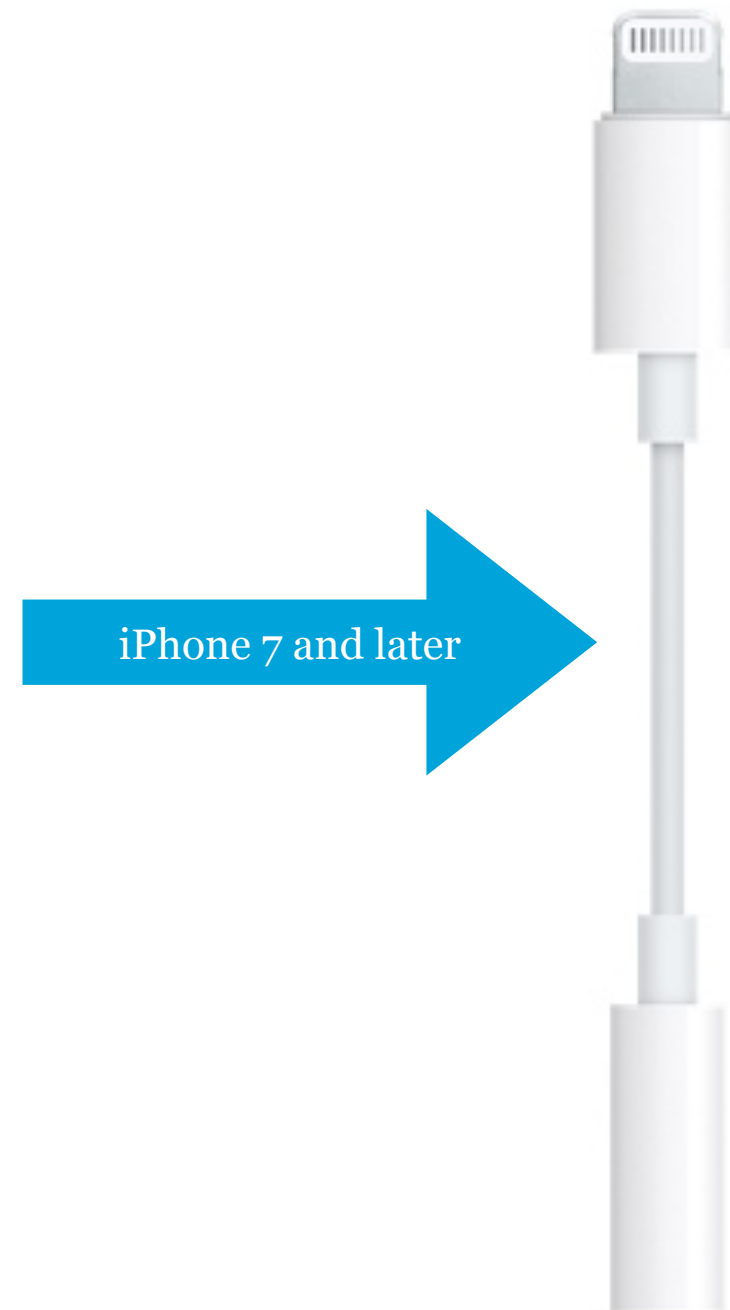


Record

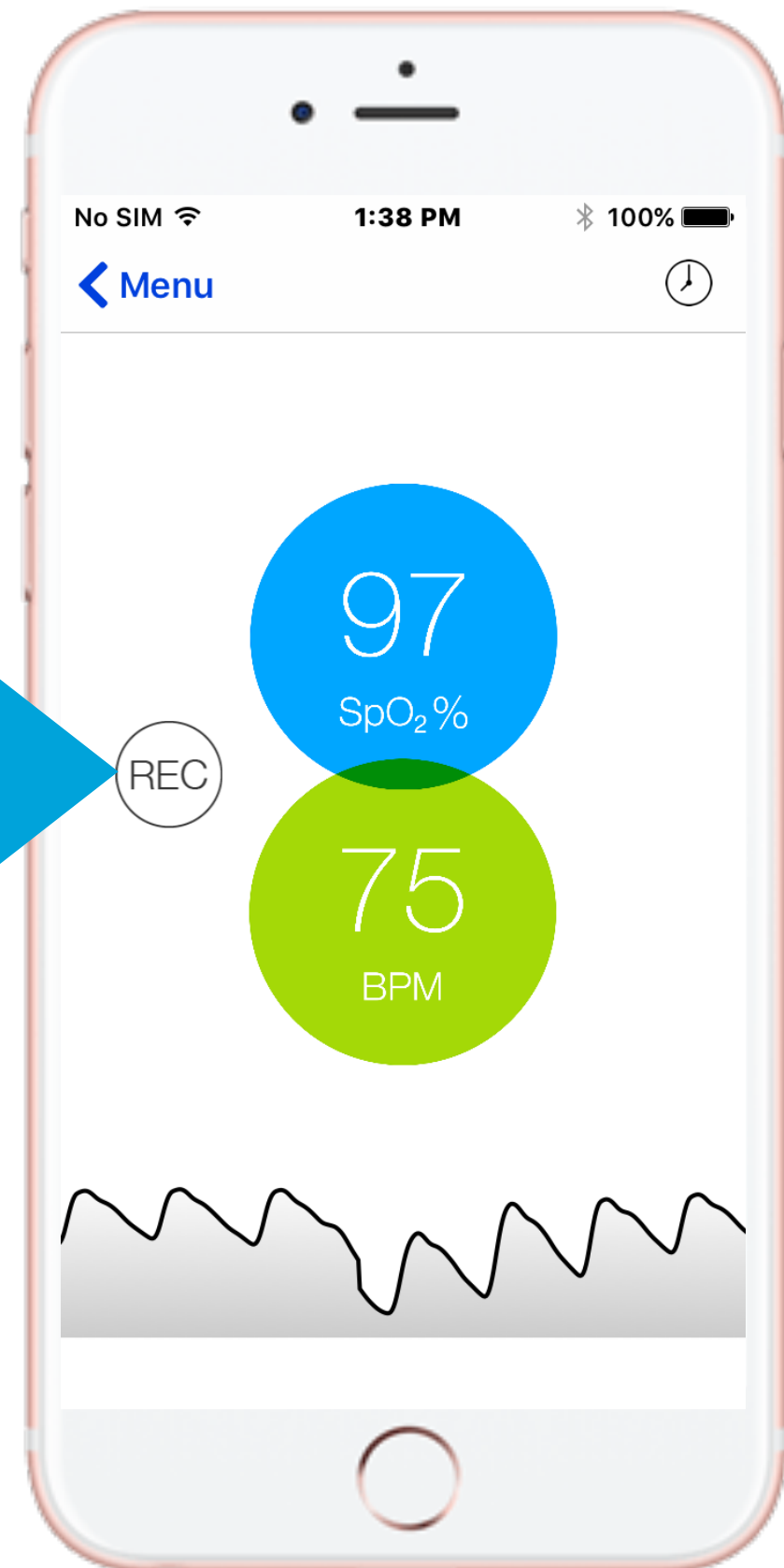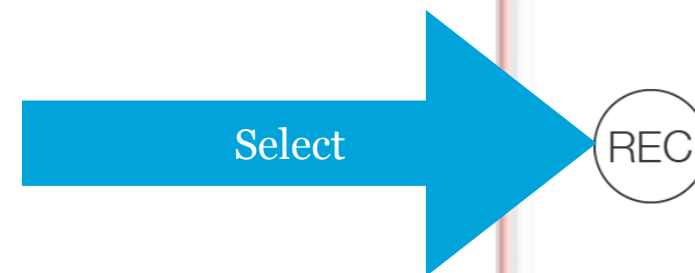


Select the REC button when it appears to initiate recording. You may turn off your phone's display now. Recording continues in the background.

History

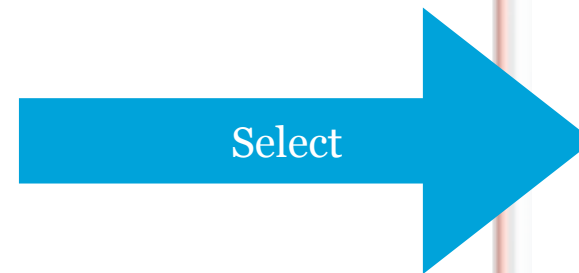


Select the History from the Kenek Edge home screen.

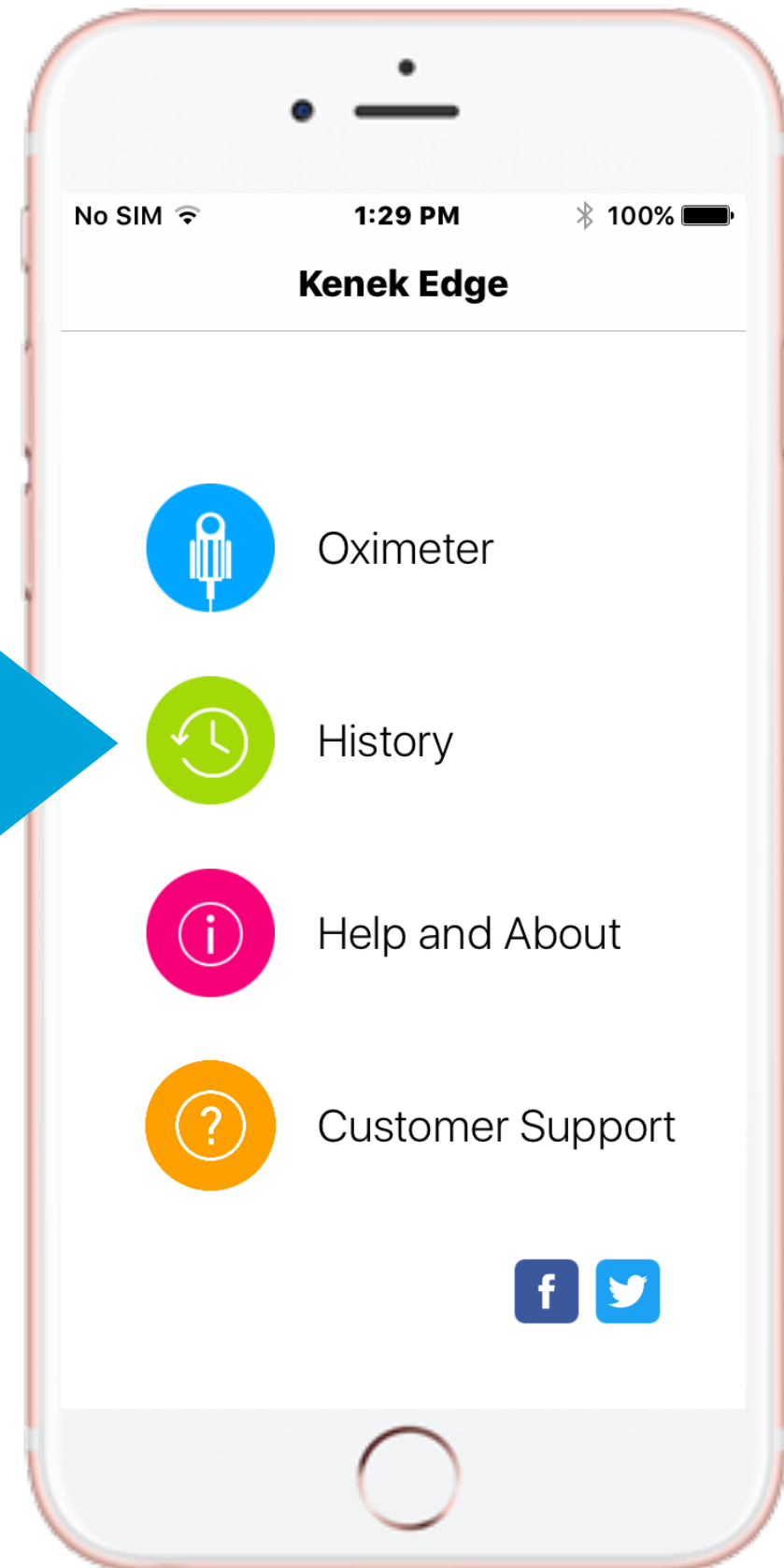

Select Recording

Select a session from your history to export by pressing the export icon. The export icon is box with an arrow pointing.

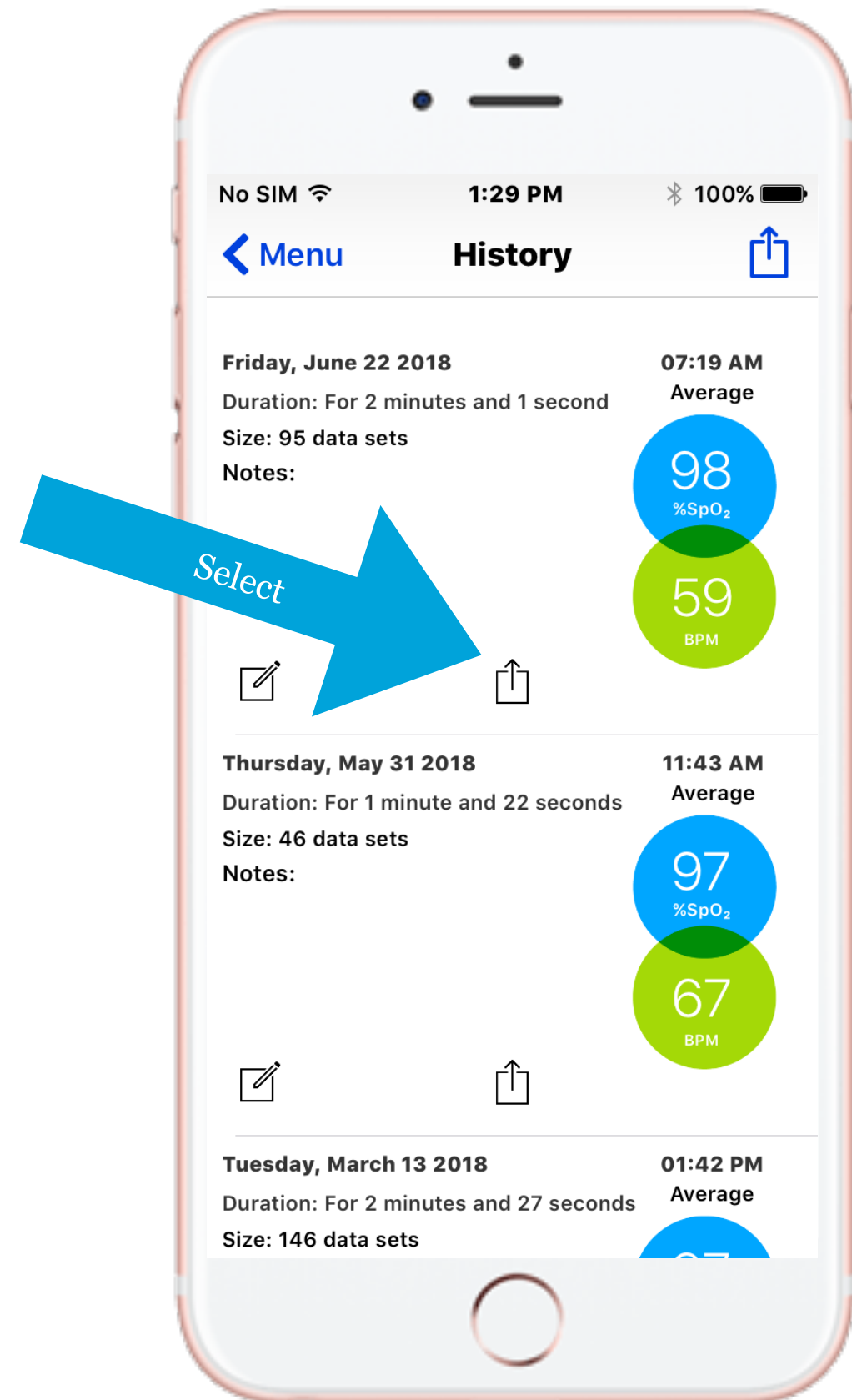


Send Email

Email the Kenek Edge export file to your computer

Kenek Edge will create the email and attach the export file as shown if your iPhone has a configured email account.

Enter an email address you receive email on your computer and press Send.

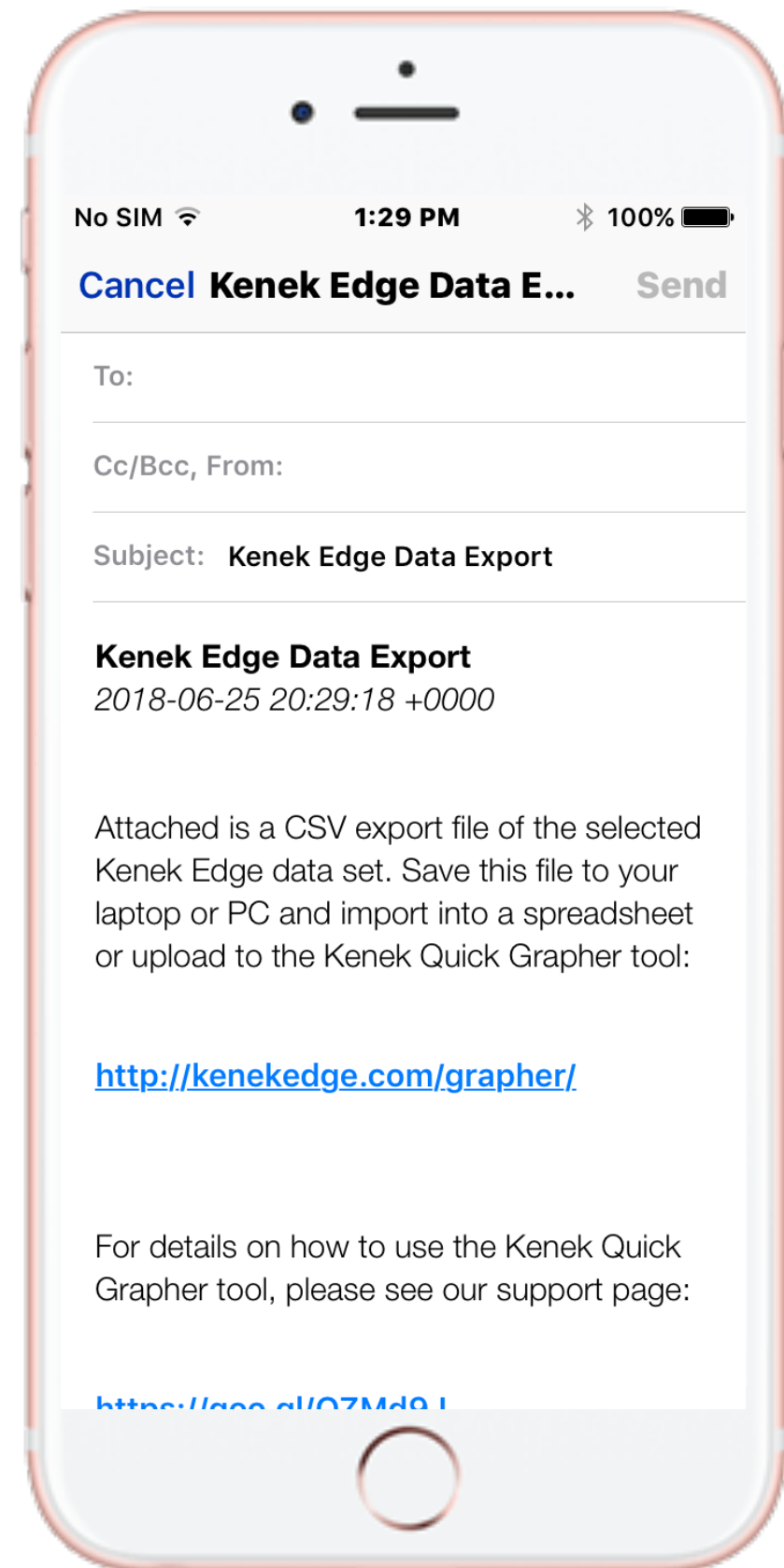


Computer

Receive Email

Save the export file to your computer

Grapher URL

Save the Kenek Edge export file to your local computer and open the Grapher web page by clicking on the URL in the email.

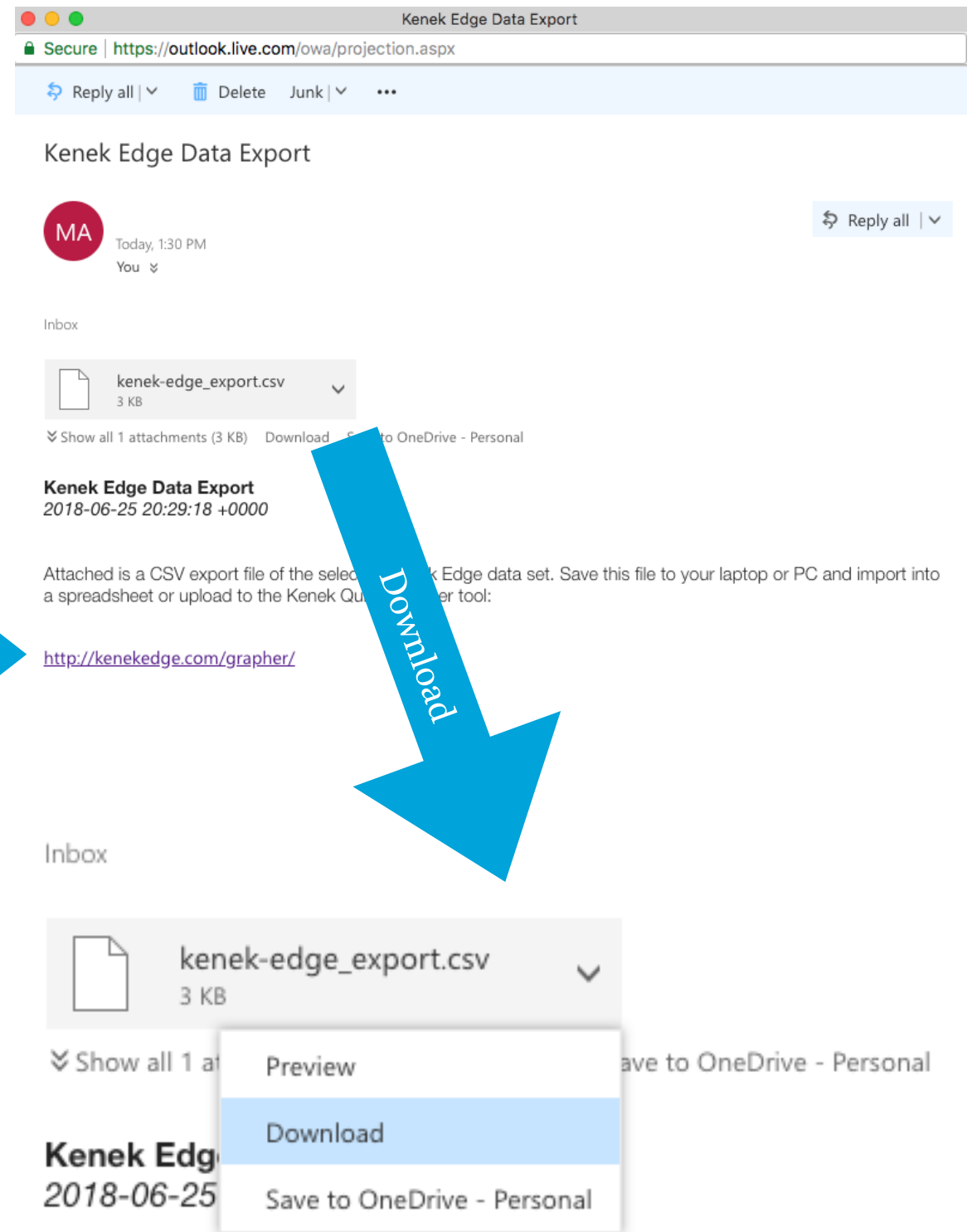


Grapher

Open the Kenek Edge export file with the Quick Grapher online analysis tool.

Select

From your computer, select a Kenek Edge export file to analyze.


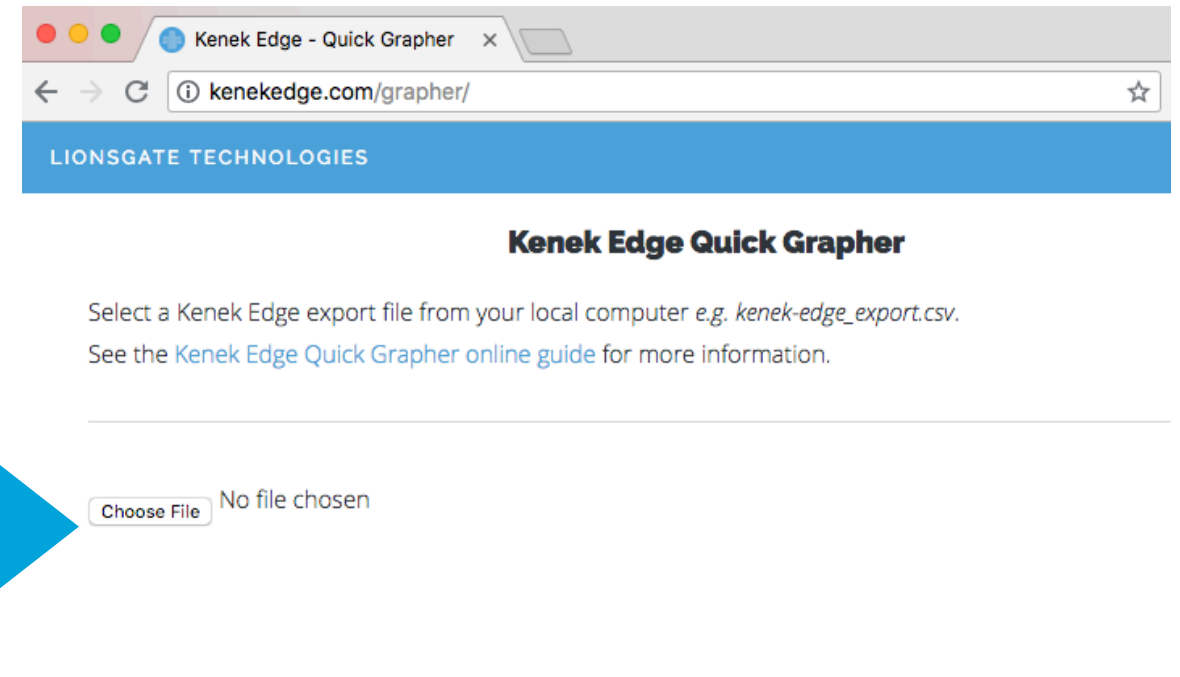


Grapher

Explore SpO₂ and heart rate data

Zoom Handle


Kenek Edge SpO₂ and Heart Rate Graphs

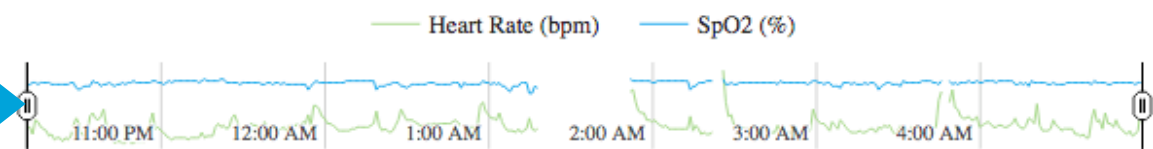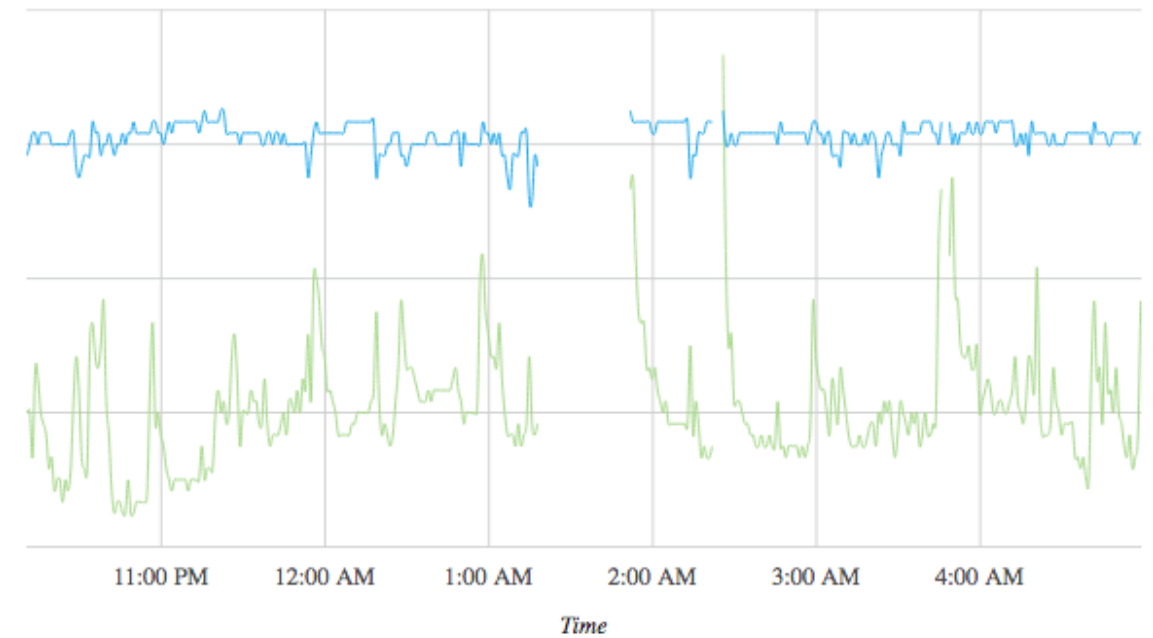


Grapher zooms into measurements you want to explore in more detail. Click on the left or right zoom handle and drag to the portion of interest. The measurements between the left and right zoom handles are shown in the chart above.

Share the report by choosing the Print Report option at the top of the Quick Grapher page (not shown).


SpO₂ Cumulative Distribution

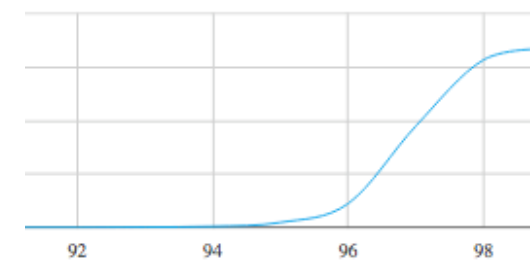

SpO₂ Probability Distribution

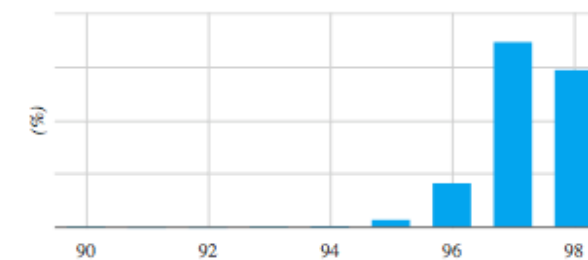

Heart Rate Cumulative Distribution

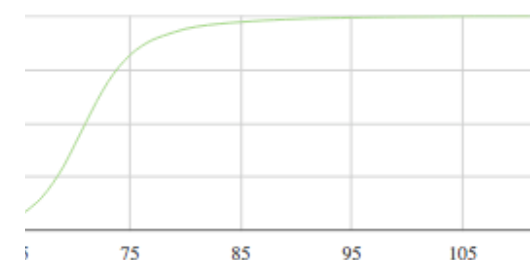

Heart Rate Probability Distribution

